

MEDICAL CARE GOCH



FFP2 Faltmaske

Made in Germany

FFP2 NR

EN 149:2001 + A1:2009
MC.2020.2

CE 2163

Bedienungsanleitung und Gebrauchshinweise



1. Entfalten Sie die Maske und halten Sie sie an den Ohrbändern mit dem Nasenbügel nach oben gerichtet.



2. Platzieren Sie die Maske über dem Kinn auf Ihrem Gesicht und befestigen Sie die Ohrbänder hinter den Ohren. Stellen Sie sicher, dass der Nasenbügel auf dem Nasenbein liegt.



3. Passen Sie den Nasenbügel mit beiden Händen Ihrer Nasenform an, um eine optimale Abdichtung und sicheren Sitz im Nasenbereich zu ermöglichen.

4. Überprüfen Sie, ob die Maske dicht an Ihrem Gesicht anliegt. Halten Sie dazu beide Hände vor der Maske und atmen Sie stoßartig aus. Bei Leckage passen Sie den Sitz der Maske und des Nasenbügels erneut an Ihre Gesichtsform an.



» Bei Beschädigung der Verpackung, die Masken nicht verwenden.



» Zum einmaligen Gebrauch bestimmt, nicht wiederverwendbar. »



Nicht waschbar.



» Nach dem Gebrauch fachgerecht in den Restmüll entsorgen. »



Bitte beachten Sie die beiliegende Gebrauchsanweisung.

» Kühl und trocken aufbewahren.

MCMedical CareGochGmbH

Haltbarkeitsdatum (M/J): **01/2024**

Produktionsdatum (M/J): **01/2021**



Vertrieb: Martin Reuschenbach Handels- und Fertigungs-GmbH & Co. KG - Siemensstraße 11-13
47533 Kleve - 02821 / 978060 - 0 - info@reuschenbach.com - www.reuschenbach.com

FFP2 Faltmaske

Made in Germany

FFP2 NR

EN 149:2001 + A1:2009
MC.2020.2

CE 2163

MEDICAL CARE GOCH



Operating instructions and instructions for use



1. Unfold the mask and hold it by the ear straps with the nose clip pointing upwards.



2. Place the mask over your chin and secure the ear straps behind your ears. Make sure the nose clip rests on the nasal bone.



3. Adjust the nose clip to the shape of the nose to create a perfect seal.

4. Check, that the mask fits tightly. To do this, hold both hands in front of the mask and exhale abruptly. In case of leakage, adjust the fit of the mask and the nose clip.



» Don't use the mask if the packaging is damaged. »



For single use only, not reusable.



» Not washable.



» Dispose properly after use.



» Please observe the enclosed instructions for use. »

Store in a cool and dry place.

MCMedical CareGochGmbH

Expiration date (M/ Y): **01/2024**

Date of manufacturing (M/ Y): **01/2021**



Vertrieb: Martin Reuschenbach Handels- und Fertigungs-GmbH & Co. KG - Siemensstraße 11-13
47533 Kleve - 02821 / 978060 - 0 - info@reuschenbach.com - www.reuschenbach.com